MOSKINTO®

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EN NOTES ON USE

Dear customer, thank you for choosing MOSKINTO. Should you have any questions, please contact our Customer Service via our website: www.moskinto.com

KEY TO THE ICONS

☞ The design of the product corresponds tot hat of a medical device of class 1.

☞ Latex-free

☞ Do not use the plaster twice.

☞ Read the instruction manual on Use carefully.

☞ Protect the plaster strips from direct sunlight.

☞ Store the plaster strips in a dry and clean place.

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USE

The plaster strips are used to relief itching and swelling after mosquito bites. Stick the Moskinto patch directly over your insect bite, do not scratch before applying. The itching should start to subside in a short time⁽¹⁾ after application. The plaster comes off automatically after 4 to 7 days.

CAUTION!

- Do not stick the plaster on open wounds.
 - In case of allergic reactions remove the patch and consult a doctor.

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CHARACTERISTICS OF THE PLASTER STRIPS

The plaster strips consist of polyester, polyurethane and acrylic adhesive. Since the adhesive coating is sensitive to heat, the plaster should be at a room temperature when applied. Moskinto is water-resistant and can be worn for several days and remains in place when taking showers or baths. Moskinto does not contain any medications or active ingredients.

Without chemical ingredients.

DISPOSAL

The packaging material is reusable. Please dispose of the packaging in environmentally friendly manner and return it tot he recycling collection. Please also dispose of the articles in an environmentally friendly manner. Contact your local municipal administration for more information.

(1) The assumption of an effect has not yet been scientifically proven. It is based on user reports and experience.

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